

New Look New Size

The AN2000 features all metal construction, including a double walled design with a powder coated exterior and stainless steel interior. Wider door (14") is designed for larger procedure trays, coiled flexible scopes and bulky instruments.




Anprolene AN2000



CYCLE MONITORING

The AN2000 Series still offers the most gentle **100% EtO sterilization** cycle on the market: **room temperature** sterilization (minimum 68° F), no steam injection, no deep vacuum, and **only 17.5 grams of gas**.

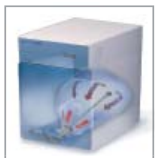

ACTIVE AERATION

This is **the most attractive new Anprolene sterilizer** available!



FREE OPERATOR TRAINING

Anprolene sterilizers now feature a number of important improvements. They include: more powerful ventilation and **purge pumps**, unique **active aeration** system, digital display **tracks cycle** progress, temperature sensor **monitors** and alerts of low temperature in the room.



OPTIONAL ABATOR

Microprocessor monitors each cycle and displays error messages and cycle progress.

AN2000 and Starter Kit



Suggested use:

Heavy duty use, wide trays or instruments, front office use.

Chamber dimensions:

20"L x 14"W x 14"H

Starter Kit - All you need for 32 cycles!

2 cases	17.5 g ampoules and sterilization liner bags / AN7916
2 pouches	Humidichip relative humidity stabilizer chips / AN1071
1 unit	Humiditube holder / AN1072
2 boxes	Dosimeter chemical indicators / AN87
2 boxes	EtO exposure indicator strips - chemical indicators / AN85
2 boxes	Steritest biological indicators / AN80
1 kit	AirScan test kit (STEL and TWA) / AN93
1 unit	Installation Kit / AN74.01

STERILIZER IS INCLUDED WITH A STARTER KIT

Features and Specifications



Unique Load Size Flexibility: The Anprolene system uses unique plastic sterilization liner bags and individual gas ampoules. We offer two different bag/ampoule combinations to match the size of your sterilization load. You never use more gas than you need.

Sterilization and Aeration in the same cabinet: The Anprolene System utilizes an advanced purge system that aerates products for two hours at the end of the cycle. Active aeration continues indefinitely until the cycle is ended and products are removed.

Free Operator Training for the Life of the Sterilizer: We recommend that all the operators take our Anprolene Key Operator Training course. Call to receive our study guide. When you are ready contact Andersen Customer Service to schedule a 20 minute test via telephone.

Proven Reliability: Anprolene is used in more than 5,000 facilities worldwide. Find out how an Anprolene sterilizer can improve your practice today. Call 800-524-3455.

Specifications:

Model:
AN2000.14


Exterior dimensions:
22"L x 17.5"W x 22.5"H

Inner dimensions:
20"L x 14"W x 14"H

Total Weight:
74 pounds (33.6kg)

Power Supply:
115 +/-10%, 50/60 Hz/150 Watts

Manufactured by:
Andersen Sterilizers, Inc.

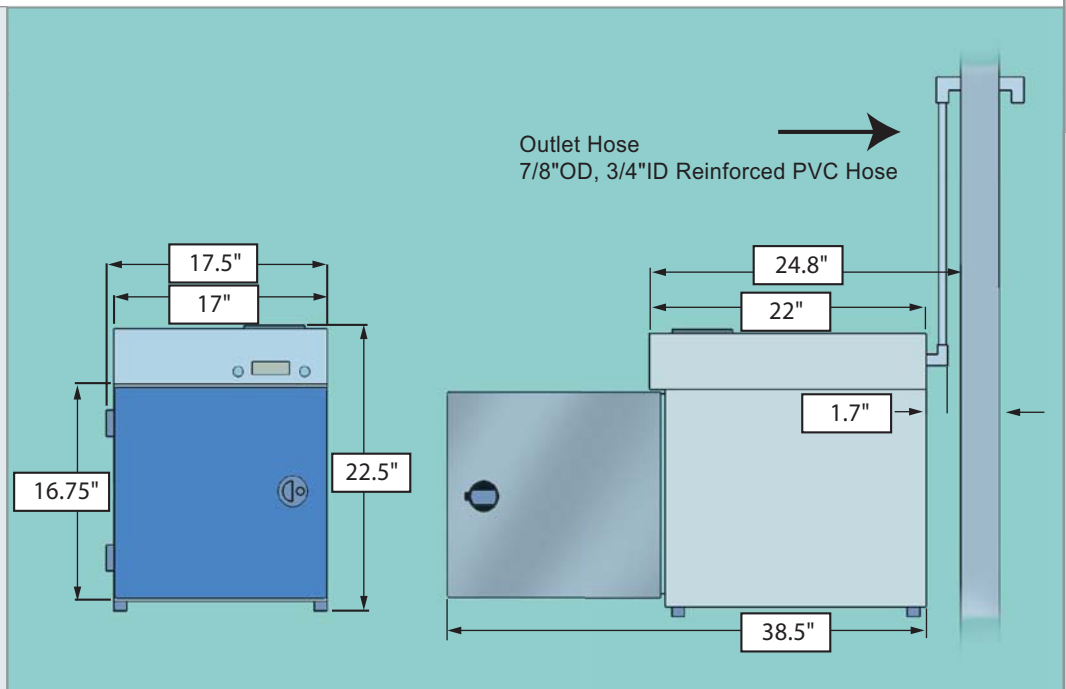

For more information about Anprolene® Sterilization Starter Kits and prices call customer service at 800-524-3455, or visit www.andcal.com

